





Check Your Blood Glucose

1. Using  choose the  icon if you are checking your BG (blood glucose) before a meal or choose the  icon if you are checking your BG after a meal.



2. Insert test strip wait for the  icon and the "Apply A Drop of Blood" message to appear.




3. Apply a drop of blood to test strip.




4. Wait for the meter to count down and display your test result.




Meter Messages

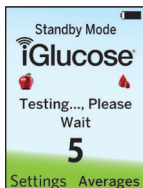

- The meter will count down from 5 to 0 seconds.

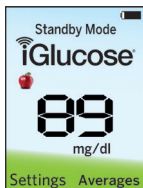

- After 5 seconds your blood glucose result will be displayed.

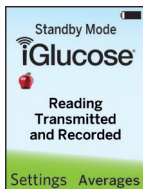

- Your reading has been sent to the cloud and you can access it on the portal.

We hope you enjoy using the iGlucose® Cellular Diabetes Care System

For assistance, please contact us at
1-844-iGlucose (1-844-445-8267)
or Email us at support@iglucose.com



iGlucose®
Blood Glucose Monitoring System



Blood Glucose Monitoring System

Quick Start Guide

Meter, Test Strips
and Lancing Device



Important Safety Instructions

- The iGlucose® Blood Glucose Meter and Lancing Device are for self-testing. Do not share them with anyone else, including family members.
- After use, all parts of the kit are considered biohazardous and can potentially transmit infectious diseases, even after following the cleaning and disinfecting procedures.
- Refer to your Owner's Booklet for more important information and detailed instructions on how to use and clean your iGlucose Meter and Lancing Device.



Intended Use

The iGlucose® Blood Glucose Monitoring System is intended to be used for the quantitative measurement of glucose (sugar) in fresh capillary whole blood samples drawn from the fingertips. The iGlucose Monitoring System is intended to be used by a single person and should not be shared.

The iGlucose System is intended for self-testing outside the body (in vitro diagnostic use) by people with diabetes at home as an aid to monitor the effectiveness of diabetes control. The iGlucose System should not be used for the diagnosis of, or screening for diabetes or for neonatal use.

Get To Know Your System

Blood Glucose Meter



- Indicates a control solution test result

mg/dL

- Unit of test result

89

- Test result



- Battery fully charged



- Battery is low and must be recharged



- Indicates a meal marker. A full apple for "Before Meal" and an eaten apple for "After Meal"



- Indicates that there is a connection to a cellular network





- Indicates when to apply a blood drop





- Indicates when to apply a control solution drop

See Owner's Booklet for additional details.

Test Strip

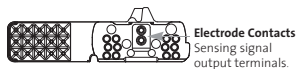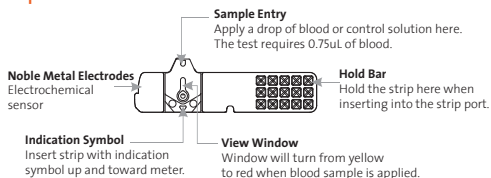

Lancing Device

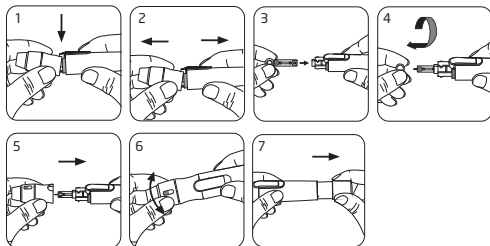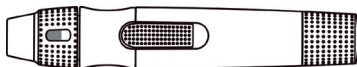



101-3GM291-022 EN(US)
MQSG01-EN Rev 2020.02