
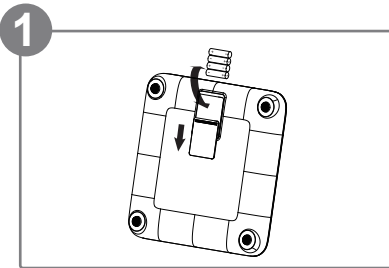



# iScale<sup>®</sup> Plus

## QUICK-START GUIDE


Cellular Weight Scale  
Model SMSC2001-G-001

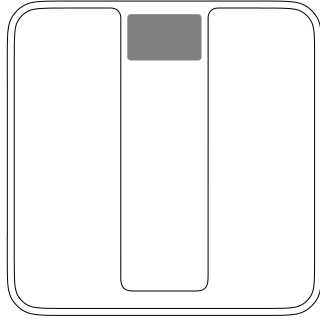


1 Insert the batteries. The device will be initialized to make sure the connection is working.

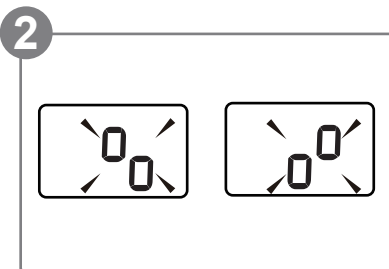

2 You will see the icons flash on the LCD screen, then the symbol and will be shown alternatively on the LCD screen, indicating that it is establishing a connection.


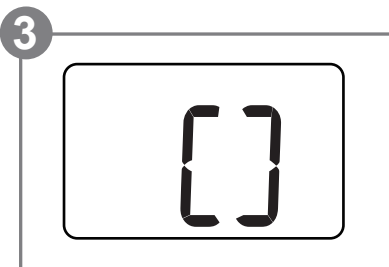


3 When symbol is shown on the LCD, this indicates the connection was successful. The device will then turn off automatically.

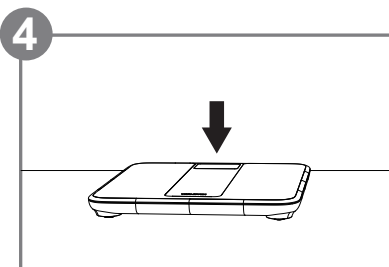

4 Lay the the scale on the floor. Please make sure the floor is flat and and is not slippery for safety purpose.

Lit #: SC-IPCQG-U-0223

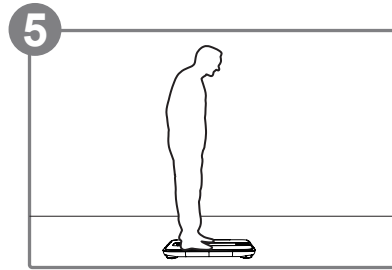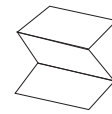

5 Step on the scale and keep stable until the weight is displayed.




6 The results will be displayed and automatically saved.




7 Data transmission will begin after measurement. The symbol blinks on the LCD to indicate data is transmitting.



8 If the data is successfully transmitted, the symbol will disappear and the LCD will display and then the device will turn off.

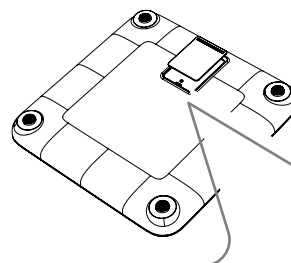

Open the battery door, you will see the black unit button down here, press it to switch units of weight.

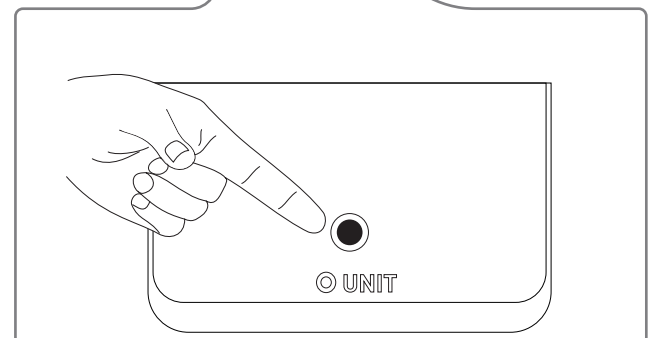

For more information, see full instruction manual or contact Smart Meter Technical Support, M-F 9 am to 5 pm ET at 1-844-445-8267 or email: [support@iglucose.com](mailto:support@iglucose.com)