








# iScale<sup>®</sup> Specifications Summary













iScale <sup>®</sup> Product Features and Information		
<b>Display</b>	Digital LCD	
<b>Battery</b>	Four AA (1.5V) batteries	
<b>Dimensions</b>	12.6 x 12.6 x 1.12 in (320mm x 320mm x 28.5mm)	
<b>Data Transmission</b>	Cellular connectivity via exclusive AT&T 4/5G CAT-M network	
<b>Scale Weight</b>	4.76 lb (2.2 kg)	
<b>Measurement Unit</b>	Pound / Kilogram / Stone	
<b>Measurement Range</b>	11 lb to 440 lb / 5 kg to 200 kg / Ost:1 1 lb to 31st: 1 lb	
<b>Accuracy</b>	<b>-Weight Measurement (lb)</b>	<b>-Weight Measurement (kg)</b>
	110 lb: +/- 0.5 lb	50 kg: +/- 0.25 kg
	220 lb: +/- 0.7 lb	100 kg: +/- 0.35 kg
	330 lb: +/- 0.8 lb	150 kg: +/- 0.4 kg
	440 lb: +/- 1.0 lb	200 kg: +/- 0.5 kg
<b>Operating Conditions</b>	Temperature: 41°F to 140°F (5°C to 40°C) Relative humidity: <=90%, Atmospheric pressure: 860 hPa to 1060 hPa	
<b>Storage Conditions</b>	Temperature: -4°F to 140°F (-20°C to 60°C) Relative humidity: <=90% Atmospheric pressure: 500 hPa to 1060 hPa	
<b>Power On</b>	Automatic, SENSE ON technology	
<b>Power Off</b>	Automatic, after 10 seconds while showing 0.0	
<b>Accessories</b>	4 AA batteries, user manual	
<b>Mode of Operation</b>	Continuous operation	
<b>Water Protection</b>	IP21 – protected against solid foreign objects >=12.5mm and vertically falling water droplets	
<b>Degree of Protection</b>	Type BF applied part	
<b>Scale Placement</b>	Flat, hard surface	
<b>Measurement Timing</b>	At least 2 hours after waking up or eating	
<b>Use Location</b>	In home	
<b>Age of Use</b>	Adults, Children (10-17 years old)	
<b>Warranty</b>	Two (2) years	



# iScale<sup>®</sup> Specifications Summary

iScale <sup>®</sup> Display Icons	
kg	Weight measurement unit - kilogram
st lb	Weight measurement unit - stone
lb	Weight measurement unit - pound
	Battery level
	Signal symbol
	Data transmitting
	Data pending to transmit
	No SIM card information

iScale <sup>®</sup> Display Errors	
	Overload - retake measurement, if issue persists check if your weight is within (11 lb to 440 lb/5 kg to 200 kg/0 st:11lb to 31 st:1lb), stop using this scale for measurement if not in range
	SIM error - contact customer support if issue persists
	No signal - move scale to a strong signal location, contact customer support if issue persists
	Hardware error - retake measurement, if issue persists contact customer support
	Data transmission error - move scale to another area (preferably closer to a window), use in an area with a strong cellular signal
	No cellular coverage - move scale to another area (preferably closer to a window), use in an area with a strong cellular signal
	Pre-loaded SIM card has been replaced or cellular service subscription suspended/terminated - contact customer support if issue persists
	Server connection error - move scale to another area (preferably closer to a window), use in an area with a strong cellular signal
	Certificate error - contact customer support
	Hardware error - retake measurement, if issue persists contact customer support