


iGlucose[®] Plus

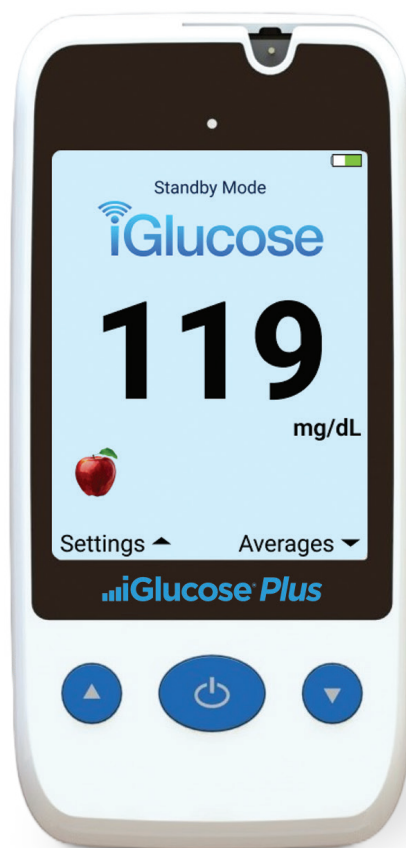


Quick Start Guide




Meter and Test Strips

Pre-Test Step

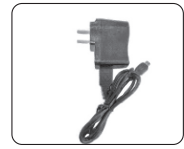
Charge the meter.



Check Your Blood Glucose

1. If using meal markers, use the home button  to choose the meal marker icon  if you are checking your blood glucose before a meal or choose the  icon if you are checking your blood glucose after a meal.




2. Hold test strip horizontally with the indicator arrow pointing down. Push the test strip straight down into the test strip port. Wait for the  icon and the “Apply A Drop of Blood” message to appear on screen.




3. Apply a drop of blood to the top of the test strip.




4. Wait for the meter to count down and display your test result. The meter sends your reading to your care team. When complete, the Reading Successfully Transmitted message appears.

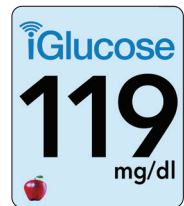


Meter Messages

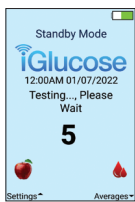

- The meter will count down from 5 to 0 seconds.

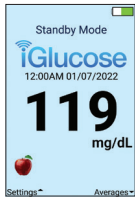


- After 5 seconds your blood glucose result will be displayed.

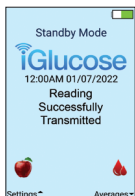





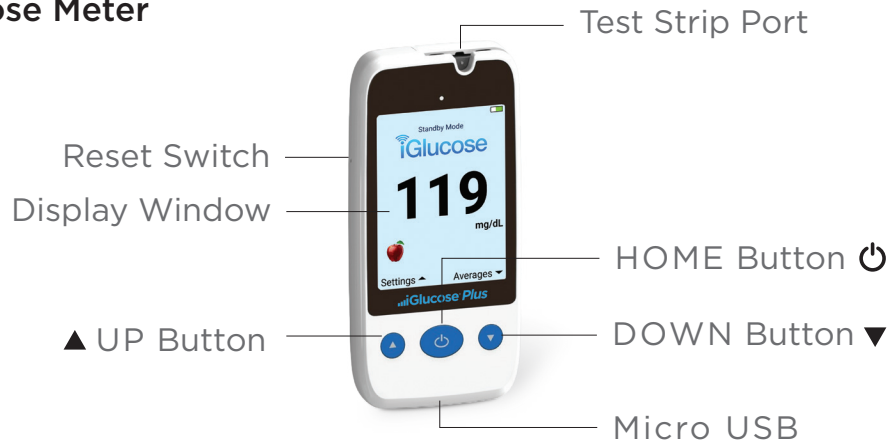


- The meter sends your reading to your care team. When complete, the Reading Successfully Transmitted message appears.

Get To Know Your System


Blood Glucose Meter



	• Indicates a control solution test result
mg/dL	• Unit of test result
119	• Test result
	• Battery is fully charged
	• Battery is low and must be recharged
	• Indicates a meal marker. A full apple for “Before Meal” and an eaten apple for “After Meal”
	• Indicates that there is a connection to a cellular network
	• Indicates when to apply a blood drop
	• Indicates when to apply a control solution drop

See User Manual for additional details

Test Strip

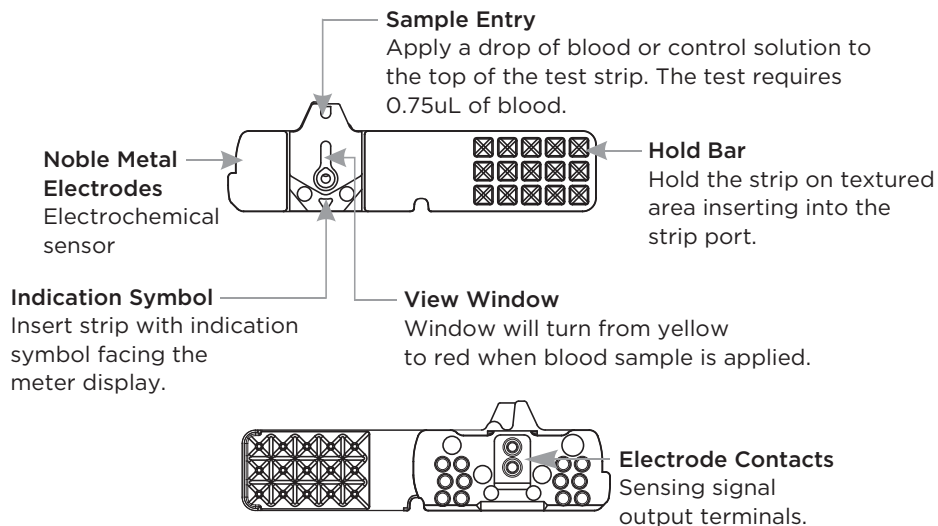

Important Safety Instructions

- The iGlucose® Plus Blood Glucose Meter is for self-testing. Do not share it with anyone else, including family members.
- After use, all parts of the kit are considered biohazardous and can potentially transmit infectious diseases, even after following the cleaning and disinfecting procedures.
- Refer to your Owner's Manual for additional important information and detailed instructions on how to use and clean your iGlucose Plus Meter.

Intended Use

The iGlucose® Plus Blood Glucose Monitoring System is intended to be used for the quantitative measurement of glucose (sugar) in fresh capillary whole blood samples drawn from the fingertips. The iGlucose Plus Monitoring System is intended to be used by a single person and should not be shared.

The iGlucose Plus System is intended for self-testing outside the body (*in vitro* diagnostic use) by people with diabetes at home as an aid to monitor the effectiveness of diabetes control. The iGlucose Plus System should not be used for the diagnosis of, or screening for diabetes or for neonatal use.

iGlucose[®] Plus Cellular Diabetes Care System Simplified. Connected. Smart.

For assistance, please contact us:
Monday - Friday 9am-5pm EST
1-844-iGlucose (1-844-445-8267)
or Email us at support@iglucose.com



iGlucose[®] Plus

Manufactured for:
Smart Meter, LLC
6206 Benjamin Rd.
Ste. 314 Tampa, FL 33634

Contact Smart Meter Customer Support
Monday-Friday 9:00AM-5:00PM EST 844-445-8267
Please contact your healthcare provider for assistance
after normal business hours.

Number:
Date:

101-3GM291-0B0 EN(US)
MQSG01-EN Rev 2024.02
Copyright 2024