

CLIENT CONTACT:
SMART METER
Erin Wojciechowski, Marketing Coordinator
Erin.Wojciechowski@iglucose.com 630-310-0624 (Mobile)

LAST REVISION DATE: March 1, 2024
FILE NAME: iGlucose_Plus_QSG_EN_030124_OPTION-2A.indd
SOFTWARE: Adobe Illustrator CC2023

10-up format

OPTION 2A
(THIS ACCORDIAN FOLDS FIRST, THEN BIFOLDS,
SO THAT FPO BARCODE SHOWS WHEN FOLDED)

101-36M291-080 EN(US)
M05G01-EN-Rev. 2024.02

I-HNSRT-QSGIGPBIL-U-0124.

Manufactured for:
Smart Meter, LLC
6206 Benjamin Rd.
St. 314 Tampa, FL 33634

Contact Smart Meter Customer Support
Please contact your healthcare provider for assistance
Monday-Friday 9:00AM-5:00PM EST 844-445-8267
Number:
Date:

iGlucose® Plus

page left intentionally blank



iGlucose® Plus
Cellular Diabetes Care System
Simplified. Connected. Smart.

For assistance, please contact us:
Monday - Friday 9am-5pm EST
1-844-iGlucose (1-844-445-8267)
or Email us at support@iglucose.com

The iGlucose® Plus Blood Glucose Monitoring System is intended to be used for the quantitative measurement of glucose (sugar) in fresh capillary whole blood samples drawn from the fingertips. The iGlucose Plus Monitoring System is intended to be used by a single person and should not be shared.

The iGlucose Plus System is intended for self-testing outside the body (*in vitro* diagnostic use) by people with diabetes at home as an aid to monitor the effectiveness of diabetes control. The iGlucose Plus System should not be used for the diagnosis of, or screening for diabetes or for neonatal use.

- The iGlucose® Plus Blood Glucose Meter is for self-testing. Do not share it with anyone else, including family members.
- After use, all parts of the kit are considered biohazardous and can potentially transmit infectious diseases, even after following the cleaning and disinfecting procedures.
- Refer to your Owner's Manual for additional important information and detailed instructions on how to use and clean your iGlucose Plus Meter.

Intended Use

Important Safety Instructions

- The meter will count down from 5 to 0 seconds.
- After 5 seconds your blood glucose result will be displayed.
- The meter sends your reading to your care team. When complete, the Reading Successfully Transmitted message appears.



1. If using meal markers, use the home button (🏠) to choose the meal marker icon before a meal or choose the 🍽️ icon if you are checking your blood glucose after a meal.
2. Hold test strip horizontally with the indicator arrow pointing down. Push the test strip straight down into the test strip port. Wait for the 🩸 icon and the "Apply A Drop of Blood" message to appear on screen.
3. Apply a drop of blood to the top of the test strip.
4. Wait for the meter to count down and display your test result. The meter sends your reading to your care team. When complete, the Reading Successfully Transmitted message appears.

Check Your Blood Glucose

Charge the meter.

Get To Know Your System

Blood Glucose Meter

Test Strip Port

Reset Switch

Display Window

UP Button ▼

DOWN Button ▲

Micro USB

mg/dL

- Indicates a control solution test result
- Unit of test result
- Test result
- Battery is fully charged
- Battery is low and must be recharged
- Indicates a meal marker. A full apple for "Before Meal" and an eaten apple for "After Meal"
- Indicates that there is a connection to a cellular network
- Indicates when to apply a blood drop
- Indicates when to apply a control solution drop

See User Manual for additional details.

Test Strip

Sample Entry

Apply a drop of blood or control solution to the top of the test strip. The test requires 0.75µL of blood.

Hold Bar

Hold the strip on textured area inserting into the strip port.

Noble Metal Electrodes

Electrochemical sensor

Indication Symbol

Insert strip with indication symbol facing the meter display.

View Window

Window will turn from yellow to red when blood sample is applied.

Electrode Contacts

Sensing signal output terminals.

iGlucose® Plus

Fabricado para:
Smart Meter, LLC
6206 Benjamin Rd.
St. 314 Tampa, FL 33634
Póngase en contacto con el servicio de atención al cliente de Smart Meter Lunes a viernes de 9:00 a. m. a 5:00 p. m. EST 844-445-8267
Comuníquese al proveedor de atención médica para obtener ayuda antes del horario comercial normal.

Numero:
Fecha:

02
13-05G01-EN-Rev 2024
(S)N71EN3-080-080-101
M05G01-EN-Rev 2024.02