


iDigiTemp

Quick Start Guide



Clean Probe

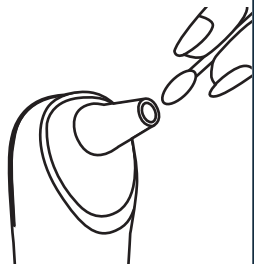

Head Mode



1

Part hair




2

Wipe sweat



3

Point at the center
(1-3cm)

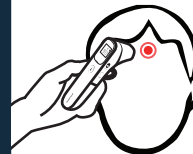


Ear Mode



1

Pull ear straight back
(Age ≤ 1 year)

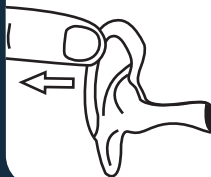

1

Pull ear back & up
(Age ≥ 1 year)



2


Insert the probe gently



Quick Start Guide





Connecting to the cellular network

When the device is turned on for the first time after battery installation or battery replacement, the device will automatically scan and connect to the cellular network. The symbol OK will appear on the screen, confirming the connection was successful.

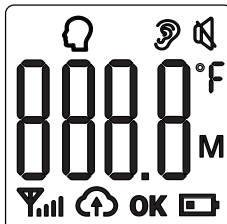


Measurement data upload

After each measurement, the measurement data will be automatically transmitted via the cellular network.

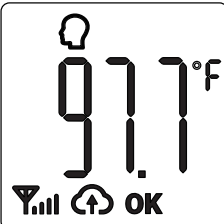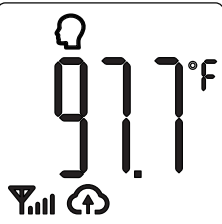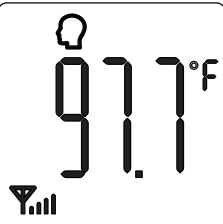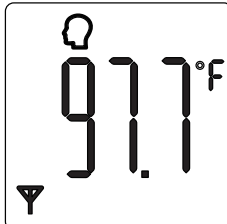

Guidance

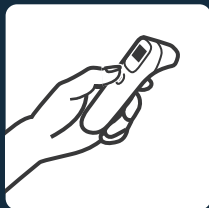

Keep Steady

Keep still during your measurement.



Repeated Use

Wait 10 seconds between each use.




Keep Forehead Clean

Make sure your forehead is clear of any hairs, sweat or other debris.



Operating Temperature

10-40 °C
50-104 °F

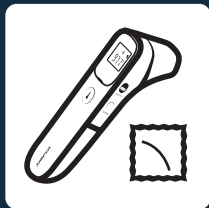

Keep Probe Clean

After continuous use, make sure to check and clean the lens.



Operating Environment

Make sure the user and thermometer are in the same area for at least 30 minutes prior to use.



Cleaning

After cleaning the thermometer, wait 15 minutes before using.




SMARTMETER™
Engaging Health Data



S-O-IDTQSG-U-0725

 www.SmartMeterRPM.com

 1-844-445-8267

 support@iglucose.com

 6005 Benjamin Rd, Suite A,
Tampa, FL 33634

..iDigiTemp

